Instruction for use

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[Intended Use]

NiTi Brush for peri-implantitis is intended for mechanical debridement and cleaning of titanium dental implant surface contaminated by osseous defect of plaque, calculus or any other foreign body materials resulted from peri-implantitis, and NiTi Brush for peri-implantitis is designed for use in dental office by dentists, dental hygienists and trained dental assistants.

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[Indication] : Peri-implantitis disease

⚠ [Side Effects] : Redness, Discomfort, Swelling

⚠ [Contraindications]

- It is contraindicated in young patient with poor attention and it can't be used in non-implant surgery patients.
- Usage of the device for pregnant, pediatric or elder patient, there is no safety information, please consult to specialist

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[Patient Population]

- Ages : adult
- Weight : Multiple
- Patient condition : The NiTi Brush is used for patients who have dental implants.

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[Intended Operator]

- Education : Expert trained who can use this medical device.
- Knowledge : Medical Knowledge
 - a) Read and understand 'westernized Arabic' numerals when written in Calibri font
 - b) Can distinguish: left arm, right arm
 - c) Understand hygiene
- Language : English or German (Languages offered in the instruction for use)
- Experience : Use by dentists, dental hygienists and trained dental assistants

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[Checklist before Use]

- Recommended temperature for operation room is 15°C~30°C.
- The reactions of patient against or incompatibility with titanium or nickel.
- Visual inspection for breakage of package.
- Visually inspect the damage, breakage, deformation, pollution from any foreign material in NiTi Brush
- Sterilization the NiTi Brush according to the appointed sterilization method
- Preparing recommended handpiece for NiTi Brush.
- Preparing sterile saline solution(NaCl) for irrigation and cooling.
- Preparing 3% hydrogen peroxide solution for the multiple use in one patient using one NiTi Brush
- Understanding of instruction for use.

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[How to Use]

- Open the outer box and pick the inner plastic box up.
- Open the inner plastic box and pick the NiTi Brush up
- Inspect the NiTi Brush for defects
- Pack the NiTi brush into the sterilization pouch
- Sterilize the NiTi Brush using autoclave.
- Attach the NiTi Brush in sterilized handpiece.
- Position the NiTi Brush to clean the targeted dental implant then clean the dental implant.
- Rinse extensively with sterile saline solution throughout the whole brushing time.
- In case of multiple use in one patient using one NiTi Brush, it is recommended to immerse the NiTi Brush in a 3% hydrogen peroxide solution and rinse with sterile saline before clean or debride another implant.

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[How to connect between NiTi brush and Dental Handpiece]

- Recommended handpiece for NiTi Brush is the CE marked rotating implant motor(handpiece).
- Any handpiece that has the deceleration rate of "20:1" is recommended.
- Recommended RPM for operation is 600RPM ~ 1,200RPM.

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⚠ [Caution]

- Only use by dentist or dental hygienist and trained dental assistants

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⚠ [Warning]

- NiTi Brush must not be used in patients reporting reactions against or incompatibility with titanium or nickel.
- Do not use the brush in the broken inner plastic box.
- In case of damage, breakage, deformation, pollution from any foreign material in NiTi Brush, do not use it.
- NiTi Brush is non-sterile product.
- NiTi Brush must be sterilized according to the sterilization method & condition of instruction for use.
- After sterilization, use the NiTi Brush immediately without re-packing or re-storage.
- Because of the risk for cross infection from patient to patient, NiTi Brush should be discarded after use.
- NiTi Brush is for single-use only. Do not re-use the NiTi Brush.
- Do not use if expiration date has been exceeded.
- NiTi Brush must be used in accordance with the instructions for use provided by the manufacturer.

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[Sterilization Method & Condition]

- Preparing the sterilization of NiTi Brush and sterile the NiTi Brush in accordance with bellow condition.
- Autoclaving Procedure

| Sterilization Method | Autoclave |
|----------------------|------------|
| Configuration | Wrap |
| Temperature | 134 °C |
| Pressure | 2.5 bar |
| Exposure Time | 15 minutes |
| Dry Time | 20 minutes |

- Keep the NiTi Brush in the sterilization pouch (or Sterilization Wrap) to keep it clean until you use it.

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[How to Disposal]

- Manage the NiTi Brush according to the rules and guidance of the disposal for disposable medical device.

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[How to Storage]

- Recommended temperature for storage is 15°C~30°C.
- Recommended humidity is 1% ~ 75%
- Keep away from the sunlight.
- Do not load or store any other materials on the NiTi Brush package.

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[Expiry date]: 5 years from manufactured date.

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◆ PICTOGRAM

| SYMBOL | DEFINITION | SYMBOL | DEFINITION |
|--------|-------------------------|--------|---|
| | MANUFACTURER | | AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY |
| | BATCH CODE (LOT NO.) | | CATALOGUE NUMBER (MODEL NAME) |
| | CAUTION | | CONSULT INSTRUCTIONS FOR USE |
| | COUNTRY OF MANUFACTURE | | DO NOT REUSE |
| | NON STERILE | | USE-BY DATE |
| | KEEP AWAY FROM SUNLIGHT | | KEEP DRY |

| | | | |
|--|----------------------------------|--|---------------------------|
| | DO NOT USE IF PACKAGE IS DAMAGED | | UNIQUE DEVICE IDENTIFINER |
| | IMPORTER | | DISTRIBUTOR |

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HANS KOREA CO., LTD.
Eunitor GmbH